

GB

For use as an intra-anal probe in males and females and as a small vaginal probe in females. For use only with Muscle Stimulation and EMG Biofeedback equipment approved to EN 60601-1 in the treatment of 'Stress', 'Urge' and 'Mixed' urinary incontinence and faecal incontinence.

For SINGLE PATIENT USE ONLY - may be re-used by the same patient.

For SINGLE ORIFICE USE ONLY - not for both intra-vaginal and intra-anal use.

Use as directed by your medical adviser.

CONTRAINDICATIONS

DO NOT USE if pregnant.

DO NOT USE with Electrical Stimulation if fitted with a CARDIAC PACEMAKER.

DO NOT USE if the package seal is not intact.

DO NOT USE if an infection, tissue damage or recent surgical scar is present.

DO NOT USE intra-vaginally during menstruation.

DO NOT USE electrical stimulation in patients with a history of carcinoma at the site of stimulation.

PRECAUTIONS

Caution is advised when using electrical stimulation on patients with altered or reduced tissue sensation.

Caution is advised during electrical stimulation if the user has a history of EPILEPSY - seek informed medical guidance.

Caution is advised when using the ANUFORM with equipment capable of delivering outputs in excess of 2mA r.m.s./cm². If in doubt contact either the equipment manufacturer or your medical adviser.

Discontinue use if an irritation or discomfort occurs - seek informed medical guidance.

SPECIFICATIONS

Material - Probe body, High Impact Polystyrene (HIPS).

Electrodes - Medical Grade Stainless Steel.

Cables - PVC over copper tinsel conductors.

Weight - 23.5g, Cable length - 30cm.

Connectors - 2 x 2mm dia. sockets.


Electrode Surface Area - 3cm² per electrode.

'In-use' product lifetime - 6 months.

CONTENTS

1 x Anal/Small Vaginal Probe


1 x Instructions

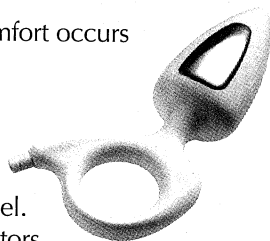

INSTRUCTIONS FOR USE

IMPORTANT

Before first and subsequent uses **always wash probe** in warm, soapy water. Rinse and dry thoroughly before use or storage. When removing probe (DO NOT PULL ON CABLES).

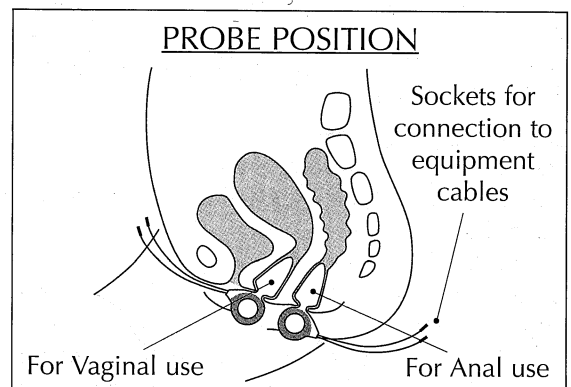

BEFORE USE

Connect equipment cables to ANUFORM. Apply a light coat of KY gel or similar lubricant to metal electrode surfaces and probe tip before insertion.

ANAL USE

Insert into ANUS whilst 'bearing down' as in passing stool. Anal sphincter should 'lock' around narrow neck of probe. 'Ring' part of probe should remain external to the anus at all times.

VAGINAL USE

Insert into vagina with electrode surfaces facing 'East-West' i.e. towards hips. Insert until body of ANUFORM is within the vaginal vault. 'Ring' part of the probe should remain external within labia.

CLEANSING AND STORAGE

Before use, always wash the Anuform in warm soapy water, rinse thoroughly and dry. **DO NOT USE BOILING WATER.** Repeat after each use and ensure that the Anuform is completely dry before storing. For additional hygiene, the Anuform may be periodically disinfected in a solution of 1% sodium hypochlorite, as used to disinfect babies' feeding bottles. Soak for 30 minutes each time, leaving the last 10 cm of cable outside the solution. Rinse thoroughly in clean water before drying and storage. Use a fresh solution each time you disinfect.

DISPOSAL

Clean thoroughly and dispose of according to local regulations.